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User Manual :  
Manual utilizador:  
Handleiding:  
Benutzerhandbuch:  
Manuale utente:

MAI le 06/01/2012

## Coussin galbé ALOVA™

Curved ALOVA™ cushions - Cojín perfilado ALOVA™ – Ergonomisch geformtes ALOVA™ Kissen  
Gewelfd kussen ALOVA™ - Cuscino anatomico ALOVA™

**CONSEILS :**

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## UK

### TIPS

The first line in bed sore prevention involves changing the pressure points of the body on the mattress and cushion an average of every two to three hours, to limit prolonged compression of blood vessels in relation to bony protrusions and to allow tissue irrigation.

### OVERVIEW:

- Cushions without pelvic support: prevention of medium to high scale pressure sore risks as validated and assessed by a clinical trial, for users without stability disorders.
- Cushions with pelvic support: prevention of stability disorders in the sagittal plane associated with a "medium to high" scale risk of pressure ulcers as validated and assessed by a clinical trial. For patients with a history of ischial or sacrococcygeal pressure ulcers.

### INSTALLATION:

- Before use, if stored below 15°C, we recommend that you expose the product for 30 minutes in a dry room (temperature > 15°C) outside of its packaging.
- Position the ischial curves and coccygeal discharge towards the rear, and the external rotator towards the front.
- Protect the cushion by using the full protective cover that is provided, making sure that the label marked "arrière/back" is positioned towards the rear of the cushion and the non-skid surface is in contact with the seat of the chair. The handle (located under the product) should be facing the front of the cushion.
- Regarding the cushion with pelvic support, make sure that it is positioned with the external rotator forward. The pelvic support should be placed into the cover slot before it is put onto the cushion
- In accordance with decree no. 2006-1637 (19/12/2006): "Service providers and equipment distributors will vouch for the proper use of the product during installation" (excerpt).

### CLEANING / DISINFECTION Promust PU Protective covering

### SECURITE :

- Make sure that the selected cushion is appropriate for the patient's weight and the size of the seat:

USER WEIGHT (min-max)	REFERENCE NUMBERS
30-90 kg	VCAG4340-VCAG3841-VCAG4041-VCAG4043-VCAG4343 + Pelvic Support Variation
30-100 kg	VCAG4341
30-110 kg	VCAG4343XL-E
30-120 kg	VCAG4540-VCAG4544-VCAG4347+ Pelvic Support Variation
30-135 kg	VCAG5044 - VCAG5044XL-E
120-160 kg	VCAG5250-VCAG5450
120-200 kg	VCAG5650-VCAG6050

### PRECAUTIONS FOR PROTECTIVE COVERINGS

- Systematically protect the cushion by using the supplied fully waterproof covers.
- Prohibit scouring, stripping or dissolving products and sharp objects in direct contact with the protective covering.
- Make sure you regularly inspect the normal wear on the protective covering and replace it if it is damaged (holes, cracks, changes in the surface, transparent when looked at in the light).

### PRECAUTIONS FOR MATERIALS

- Avoid prolonged damp contact with the foam.
- Carry out regular visual checks on the condition of the foam: the ALOVA material visibly setting or a non uniform and very slow return are all signs of ageing which compromise the support properties which help to prevent pressure ulcers.

### STORAGE

- ALOVA supports should preferably be stored flat, away from direct sunlight and humidity.

### DISPOSAL

- Do not burn or dump the support and its protective covering outside areas intended for this purpose.

### WARRANTY

The ALOVA (overlay) mattress is guaranteed for 3 years. This guarantee starts on the date of purchasing the product from your distributor.

In the event of single patient usage, the guarantee covers the support and its protective covering. In the case of multi-patient usage, we recommend replacing the protective covers every 2 years. This guarantee does not cover normal wear on the product and its protective covering and does not substitute legal guarantees. Please contact YOUR DISTRIBUTOR and show him your faulty product. They will carry out the necessary procedures with our company in order to carry out repairs or make an exchange

### PRODUCTS LIFETIME :

- Estimated lifetime of an Alova viscoelastic foam cushion is 4 years

### COMPOSITION OF THE CUSHION :

- The cushion is 100 % moulded polyurethane viscoelastic foam.

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