

DEPENDING ON THE PATIENT'S CLINICAL CASE

RISK OF PRESSURE ULCERS
LOW TO MEDIUM¹


RISK OF PRESSURE ULCERS **MEDIUM TO HIGH**¹

RISK OF PRESSURE ULCERS
HIGH TO VERY HIGH¹

Bed-ridden²
≤ 15 hours/ Day


- Patient who mobilizes by himself with difficulty,
- Without significant neurological disorder,
- Without arterial disease,
- Patient in good general condition²

STATIC MATTRESS

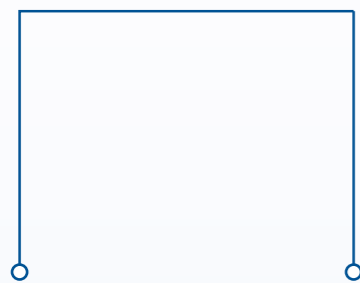


HIGH-RESILIENT FOAM

- MORFÉA**[®] 30 ≤ 120 kg
- APLOT**[®] 40 ≤ 120 kg
- EPSUS**[®] 30 ≤ 120 kg

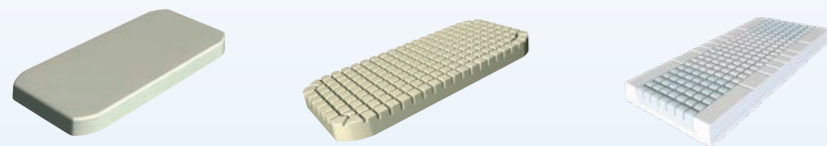


Bed-ridden² **> 15 hours/ Day**

PATIENT UP DURING THE DAY² (Frequency of position changes⁵)

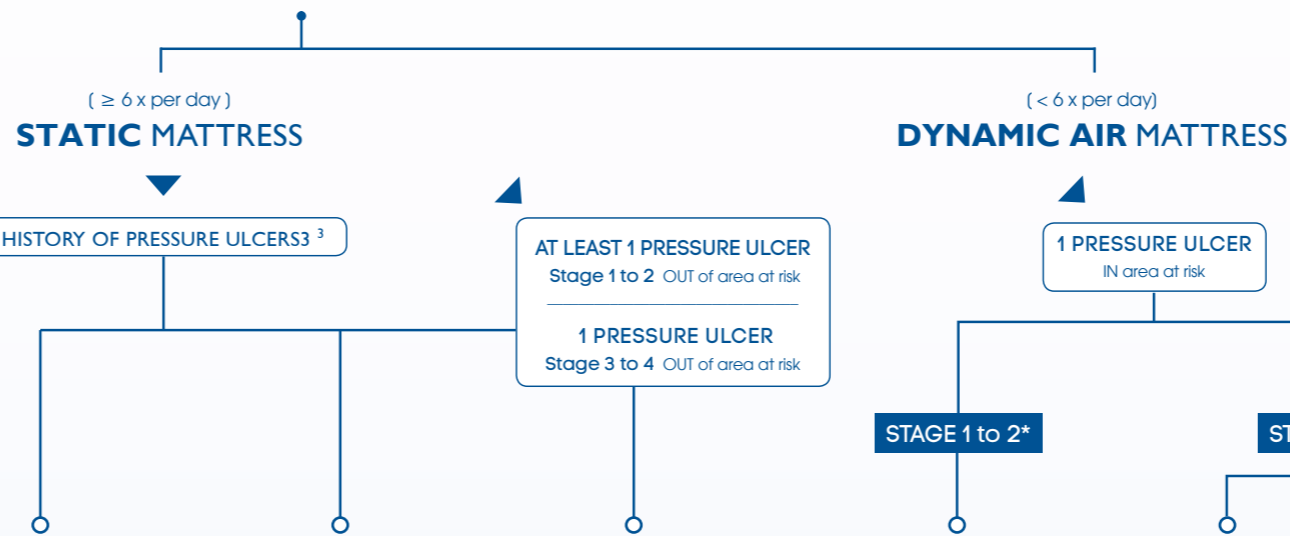

VISCOELASTIC FOAM

- ALOVA**[®] VISCOELASTIC³ 20 ≤ 180 kg
- VISKOA**[®] 30 ≤ 150 kg
- ALOVA**[™] XXL⁴ 135 ≤ 270 kg

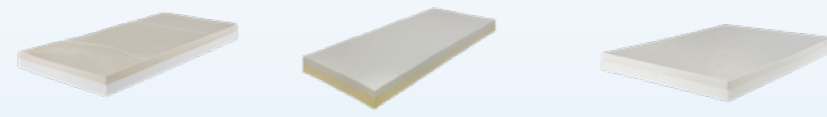

AUTOMATIC

- oxtair one[®] plus** 30 ≤ 150 kg
- oxtair automorpho[®] plus** 30 ≤ 165 kg
- oxtair automorpho[®] cen sor[®]** 30 ≤ 165 kg, 30 ≤ 180 kg, 30 ≤ 200 kg



Bed-ridden²
24/7

Patient in poor general condition, and/or with an arterial disease, and/or a recent severe neurological disorder²

DYNAMIC AIR MATTRESS

SEVERAL PRESSURE ULCERS
Stage 1 to 2 IN area at risk
WITHOUT TAD - IF BED-RIDDEN 24/7: AT15

SEVERAL BED SORES
Stage 1 to 4 IN area at risk
WITHOUT TAD - IF BED-RIDDEN 24/7: AT15

POOR GENERAL CONDITION

Pressure ulcer(s) OUT of AREAS AT RISK or with possible exclusion of the support: use of localized relief devices
Pressure ulcer(s) IN AREAS AT RISK: use of positioning systems and intervention of caregivers 3 times a day for verification and turning the patient over

- 1: According to validated scale and clinical opinion
- 2: According to the opinion of the CNEDIMTS from 12/22/2009
- 3: VMA62 weight ≤ 120 kg; VMA63 weight ≤ 150 kg; VMA64 weight ≤ 180 kg
- 4: Different versions of the ALOVA mattress are available: with stabilizing lateral edges, XL, DUO and pediatric versions
- 5: EPUAP-NPUAP-PPPIA recommendations, 2014



RISK ASSESSMENT ACCORDING TO CLINICAL OPINION AND VALIDATED SCALES

RISK	LOW TO AVERAGE	AVERAGE TO HIGH	AVERAGE TO HIGH
BRADEN SCALE	Score from 17 to 13	Score from 12 to 8	Score < 8
WATERLOW SCALE	Score from 10 to 15	Score from 15 to 19	Score from 20 and up
NORTON SCALE	Score from 14 to 12	Score from 11 to 9	Score from 8 to 5

Photos not contractually binding

These products are regulated Class 1 medical devices that bear the CE marking under this regulation. Read carefully the product instructions before use.

