## Matelas en mousse STRAT'AIR 2

## CONSEILS

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## STRAT'AIR 2 Foam Mattress

## TIPS

The first prevention of the pressure ulcers to the bed consists in changing the points of support of the body on the mattress on average every three hours to limit the prolonged compression of the vessels to bony projections and to allow the irrigation of tissues.  
Do Not overtighten the bed linen thus reducing the effective support of the mattress.

## INDICATIONS

- Medium to high risk of pressure ulcers evaluated according to the accepted clinical scales and professional judgement.
- Established pressure ulcers: follow medical recommendations.

## INSTALLATION

• Position the mattress so that the label sewn into the cover is situated at the head end. The pyramid-shaped stratum of the mattress should be on the underside.

## CLEANING / DISINFECTION

- Promust PU and Promust PU HD Protective covering

Item treated with a biocidal substance that is safe for the user. www.winnicare.fr

## SAFETY

- Limit it to users weighing between 30 and 120 kg.
- Systematically protect the mattress by means of a protective covering which is always delivered dry: if the foam has repeated contact with fluid, its properties will be modified regarding supporting the body uniformly and constantly as well as helping to lower pressure values.
- Prohibit scouring, stripping or dissolving products and sharp objects in direct contact with the protective covering.
- Make sure you regularly inspect the normal wear on the protective covering and replace it if it is damaged (holes, cracks, changes in the surface, transparent when looked at in the light).

## Precautions for materials

- Avoid prolonged damp contact with the foam.
- Carry out regular visual checks on the condition of the foam: the material visibly setting or a non uniform and very slow return are all signs of ageing which compromise the support properties which help to prevent pressure ulcers.

## STORAGE

- Supports should preferably be stored flat, out of direct sunlight and excessive humidity.

## DISPOSAL

- Do not burn or dump the support and its protective covering outside areas intended for this purpose.

## GUARANTEE

The STRAT'AIR 2 mattress is guaranteed for 5 years.  
This guarantee starts on the date of purchasing the product from your distributor. In the event of single patient usage, the guarantee covers the support and its protective covering. In the event of multiple patient usage, the length of the guarantee for the protective covering is reduced to 1 year. This guarantee does not cover normal wear on the product and its protective covering and does not substitute legal guarantees. Please contact YOUR DISTRIBUTOR and show him your faulty product. They will carry out the necessary procedures with our company in order to carry out repairs or make an exchange.

## LIFETIME OF PRODUCT

- The estimated lifetime of the multiple strata foam is 5 years.

## COMPOSITION OF PRODUCT

- 100% High Resilience Polyurethane Foam.

## SUPPLEMENTARY TECHNICAL INFORMATION

Maximum surface pressures (mm Hg) for indicated patient weight.  
Measured on a flat (0° tilt) mattress

Reference	Max. Weight	Head	Pelvic	Heel
Mattress + Promust PU Cover	80 kg	51	37	51

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