**COJINES ANTI-ESCARAS - SIGEL**

Almofadas antiescaras - Sigel

Liquid flotation cushions - Sigel

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Congratulations on your purchase of SIGEL liquid flotation cushion, designed and made so that you enjoy ideal performances, comfort and personal satisfaction.

The SIGEL cushions for pressure ulcers provide a larger contact area, which reduces the pressure per unit of area, one of the factors that influence the appearance of pressure ulcers. They also provide refreshment thanks to the special liquid inside.

Every cushion SIGEL is made of an internal part and a cover. The superior part of the cover is made with a fabric which complies with the norm JOINS IN 1021-1 & 2 and the lower part is made of an anti-slip fabric that prevents the cushion from moving in the seat meanwhile is being used.

The cushion SIGEL complies with the norm prEN 12182: "Technical helps for persons with disability. General Requirements and test methods", to the New Regulation (EU) 2017/745 for Medical Devices.

## WARNINGS AND RECOMMENDATIONS:

- Ensure that the patient's weight is lower than the maximum weight that the cushion can support.
- Check that the dimensions of the cushion are adapted to the wheel-chair.
- Place the cushion on the wheel-chair with the anti-slip fabric at the bottom.
- Eliminate from the superior part of the cushion the possible creases that could appear.
- Check regularly the state of the internal part and of the cover after several washings.
- Avoid the accumulation of humidity in the cover.
- Avoid putting the internal part close to any fire sources.

**GUARANTEE:** Your SIGEL liquid flotation pressure ulcer cushion has been rigorously inspected; on the label of the internal part you will find a serial number which relates the above mentioned cushion to the materials used and the manufacturing processes. This cushion is guaranteed for 1 year, under normal conditions of use and respecting the recommendations for use, as well as an estimate of minimum useful life of 1 year. In the event of a defect on the cushion, please contact with the distributor where it was purchased, with the proof of purchase, so that the cushion may be inspected and, if it is necessary, in accordance with the manufacturer, be replaced or repaired. The distributor and Winnicare shall be excluded from any responsibility in the event of imperfections as a result of misuse or failure to adhere to the recommendations given in these instructions. (Puncture, rip, etc...) In accordance with Council Directive 85/374 / EEC of July 25, 1985, on the approximation of the laws, regulations and administrative provisions of the Member States regarding liability for damage caused by defective products, the distributor and / or WINNCARE will not be responsible for the damage caused by the defect that did not exist at the time they put the product into circulation or that this defect appeared later

**MAINTENANCE:** For hygienic reasons it is recommended that the cushion is always used by the same person. The internal part can be washed by introducing it in a container filled with water and neutral soap, proceeding to its brushing and later rinse with water. The drying will be done with a cloth or towel or letting it dry to the air. It should be done in horizontal position in a fresh place, avoiding the exhibition to the solar beams. Do not store on sharp objects or sharp edges that could damage the cover or the internal part. In case of making a puncture in the internal part, dry and clean with alcohol the area around the puncture and put a few stickers of cello tape for soft plastic. To wash the cover follow the instructions of the label sewn to it.

**ELIMINATION:** The internal part of the cushion is made of PVC and polyurethane foam. It therefore must be disposed of in accordance with recycling ordinances or local recommendations regarding recycling.

102901001 / UDI: 8436571820358	42x42x6 cm.	max. 120 Kg.
102901002 / UDI: 8436571820365	42x42x6 cm.	max. 120 Kg.
102902001 / UDI: 8436571820372	42x42x4,5 cm.	max. 100 Kg.
102902002 / UDI: 8436571820389	42x42x4,5 cm.	max. 100 Kg.
102903001 / UDI: 8436571820396	41x41x3,5 cm.	max. 80 Kg.
102903002 / UDI: 8436571820402	41x41x3,5 cm.	max. 80 Kg.

Servicio de asistencia técnica  
 Serviço de ajuda técnica  
 Technical support service  
 Kundendienst - Stap van vrijgeven technicus

[sat@winncare.es](mailto:sat@winncare.es)

# WIN CARE

Carretera Masía del Juez, 37  
 E 46909 · TORRENT  
 Valencia · Spain  
 Tel. +34 96 156 55 21  
[www.winncare.es](http://www.winncare.es)  
[ventas@winncare.es](mailto:ventas@winncare.es)

LICENCIA SANITARIA Nº 5191 P.S.